

## CORFLEX IS A GLOBALLY BASED

# MANUFACTURER AND DISTRIBUTOR OF QUALITY

## ORTHOPEDIC AND REHABILITATION PRODUCTS



## **ELBOW SPLINTS**

## **XR CUBITAL TUNNEL SPLINT**

- Innovative support that immobilizes and positions the elbow at 122° to help reduce pain and numbness associated with Cubital Tunnel Syndrome
- Indicated for Cubital Tunnel Syndrome, post-op application or post-cast removal
- Constructed of extra rigid A.B.S. material with medial contouring on shell and plush cloth liner (replacement liners also available)



#### PRODUCT NUMBER

S/M	M/L	L/XL
37-2101	37-2192	37-210

## SIZING GUIDE

S/M	Fits most females & smaller framed males
M/L	Fits most males & larger framed females
L/XL	Fits large framed males & XL framed females

## APPLICATION INSTRUCTIONS

- 1. Locate medial side of splint as marked.
- 2. Apply device with arm flexed, following the contour of the splint and making sure medial sticker is on the inside of arm.
- 3. Attach middle strap by wrapping it around the entire unit and attaching to itself.
- 4. Slide bottom strap through buckle and attach at desired comfort level.
- 5. Complete application by sliding top strap through buckle and attach at desired comfort level.

#### CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Wipe plastic shell with damp cloth, you may use disinfectant or mild soap, and wipe dry.

For products with a removable liner, you may hand wash liner in warm water with mild soap, air dry, do not use high temperatures to wash or dry.

# **HOW TO PLACE AN ORDER**

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:

